REGENYAL
idea

Lips

CROSS-LINKED
HYALURONIC ACID

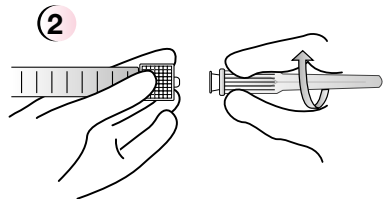
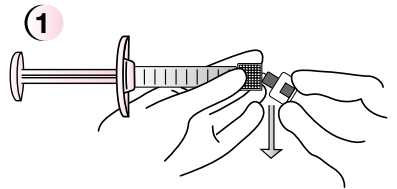
≈ REGENYAL
LABORATORIES

REGENYAL
idea

Lips

CROSS-LINKED
HYALURONIC ACID

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The product is only to be used by medical staff as described in this information leaflet.

DESCRIPTION

Regenyl Idea Lips is a reabsorbable medicinal product (physiological, apyrogenous sterile gel) to be used as a filler for the correction of medium and deep facial cutaneous sagging and to increase the volume and contours of the lips. The principle component is cross-linked Hyaluronic Acid of non-animal origin, produced by bacterial fermentation.

COMPOSITION

Cross-linked hyaluronic acid 25 mg/g
Phosphate buffer, Water for injectable solutions 1 g

PACKAGE

- 1 ml single dose

Each box contains:

- information leaflet
- sealed blister pack containing a pre-filled sterile single use syringe
- adhesive labels to stick to the patient's medical records for product identification
- two sterile needles (27 G Sterimix CE 0120 - 30 G Sterimix CE 0120).

MODALITY OF ACTION

Regenyl Idea Lips must be injected into the cutaneous tissue for the supplementation of the extra-cellular root and the intra-cutaneous volumetric increase, that is, for the treatment of cutaneous depressions caused by wrinkles and scarring.

AREAS OF USE

Regenyl Idea Lips is an implantable medicinal product created in conformity with Regulation 93/42/CEE MDD, designed for the treatment of imperfections in the following areas of the face:

- nasolabial folds
- wrinkles around the mouth
- facial and acne scars
- glabellar wrinkles
- wrinkles on the forehead
- areas of the face which require enrichment of the facial tissue (cheeks, chin, cheekbones, lips) with temporary volumetric increase through growth of the soft tissues.

The outcome of the treatment depends on the type of skin and the nature of the imperfection: the less evident the imperfection, the better the result.

INSTRUCTIONS FOR USE

Before any form of intervention with Regenyl Idea Lips, the doctor must carry out an adequate case history and a comprehensive assessment of the conditions of the patient, in order to guarantee the total absence of contraindications to the implant.

The areas to be treated must be identified and evaluated bearing in mind the criteria relative to lines and symmetries to follow.

Local anaesthetic may be used for treatment to the lips, in order to guarantee the necessary comfort to the patient.

The doctor must inform the patient in advance of the modalities of the intervention, its nature, warnings, precautions and possible individual outcomes, potential adverse responses, the expected duration of the implant and the eventuality of a supplementary intervention for the maintenance and/or definition in detail of the achieved result.

The area of intervention must be cleansed with antiseptic solution before proceeding to the implant.

Extract the syringe from the blister, remove the top as shown in the figure and screw the needle or the cannula to the luer-lock, complete with protection.

Only remove the cover just before performing the procedure.

Regenyl Idea Lips is administered with a sterile needle (27 G - 30 G) in compliance with the Luer-Lock standards with normalised connections, which is included in the pack.

The implantation must be carried out in correspondence to the medium/deep derma; the procedure is, however, at the discretion of the doctor and depends on the correction desired and the method of implantation

adopted.

At the end of the procedure it is advisable to carry out a delicate massage of the treated area in order to allow for better distribution of the implant.

DOSE AND ADMINISTRATION

The volumetric grading printed on the syringe has an indicative value: it is left to the doctor's discretion to choose the dosage to be used for each individual case.

The intervals at which the treatment should be repeated depend on various factors, regarding both the physiology of the patient (type of skin, individual metabolism, anatomy, age) and their lifestyle. Another element to consider is relative to the implant techniques used. For maintenance of the obtained results it is advisable to repeat the treatment periodically, every 6-12 months.

PRECAUTIONS

During the treatment the usual precautions in the case of percutaneous procedures must be adopted. The risks are the common ones of infection relative to the type of treatment.

Regenyl Idea Lips must not be used on patients affected by:

- infective or inflammatory processes near the area of intervention
 - known hypersensitivity to cheloids
 - allergy to the components
 - immune system illnesses
 - serious pathological skin conditions
 - problems with coagulation or anti-coagulating therapies in progress.
- Close to treatment the patient should avoid consumption of substances which act on blood fluidity (aspirin, NSAID, Vitamin E), in order to minimise the possibility of bruising or bleeding of the injected areas.
- Regenyl Idea may not be implemented in areas where there are mammary, tendon, bone or muscular implants present.
- Following the treatment, and until the complete absorption of swelling and reddening, the implanted areas must not be exposed to excessive heat (sun, UV tanning sessions or laser) or intense cold.

After use, needles and syringes must be disposed of according to the procedures indicated for hospital waste.

ADVERSE REACTIONS, WARNINGS AND CONTRAINDICATIONS

As with any percutaneous injection, there can be incidents of:

- inflammatory reactions (reddening, edema, etc), sometimes associated with itching and pain to the touch
- hematoma
- hardening or nodules at the point of injection
- colouring or discolouring of the skin at the point of injection

These reactions usually disappear within a few days. If they persist for more than a week the patient should contact their doctor immediately.

Regenyl Idea Lips is an intra cutaneous injection and must not be injected into blood vessels.

There are no reports of problems due to excessive doses or interaction with other medicines.

Do not use in pregnancy.

Regenyl Idea Lips is available in single dose sterile packages. The product must not be reused for future applications or on other patients. Once open the product must be used immediately.

Any remaining product must be disposed of.

Do not use the product if the packaging is damaged.

Do not mix with other injectable substances and do not carry out other implants together with Regenyl Idea Lips.

The implant must be carried out in an aseptic environment observing the appropriate techniques.

Fill in the adhesive label contained in the box and stick it to the patient's medical records kept at the doctor's surgery.

Keep out of the reach of children.

In rare cases, the product can cause allergic reactions.

STORAGE

Regenyl Idea Lips must be stored at a temperature between 2° and 28°C.

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STERIMEDIK LTD
REGENYAL LABORATORIES Srl